# VACO<sup>®</sup>ankle

vacuum  
technology

## Bedienungsanleitung Instructions for use Mode d'emploi Instrucciones para el manejo

# OPED

### OPED GmbH

Medizinpark 1  
83626 Valley/Oberlaindern  
Germany

Fon +49 (0)8024/60 81 82-10  
Fax +49 (0)8024/60 81 82-99

mail@oped.de  
www.oped.de

### OPED AG

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### E english

With VACO<sup>®</sup>ankle, you get an ankle splint that ideally combines stability and wearing comfort. A self-adapting vacuum system combined with a stable lattice frame structure lets you walk normally with maximum protection given to your ankle's injured ligaments. What's more, the vacuum cushion allows optimal customized adaptation to all kinds of swelling conditions without causing any bruising.

**Indications: Injuries to the ligaments of the ankle joint**

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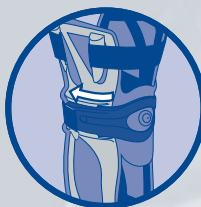
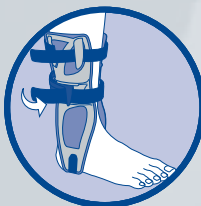
### 2 Von hinten anlegen Attach from behind

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VACO®ankle anlegen  
Applying VACO®ankle

Application de VACO®ankle  
Colocar el VACO®ankle

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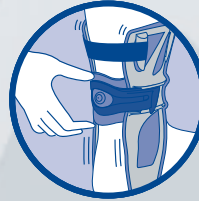
## English

### Safety tips

To ensure optimum support VACO<sup>®</sup>ankle should be worn inside a laced shoe. Thus, a more effective way of limiting the lateral bending potential of the foot can be accomplished without restricting the normal walking gait.

Wearing a sock underneath is recommended as it improves comfort and hygiene.

### Cleaning and care

Wash your VACO<sup>®</sup>ankle by hand with lukewarm water and a mild detergent. Use a cloth to dry the splint or let it dry at room temperature.

Like all our products, VACO<sup>®</sup>ankle is recyclable.

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**OPED Inc.**