

OPED

Vacuum system for diabetic feet



VACO[®]ped
diabetic foot

The VACO[®]ped diabetic foot system is an orthopedic treatment concept for the diabetic foot syndrome similar to a total contact cast. By utilizing a vacuum cushion (bean bag) in conjunction with a dampening insole and a rocker bottom, the product has been designed to evenly distribute the pressure underneath the foot. Thus, pressure peaks can be reduced in order to promote wound healing or to prevent wounds from occurring. The VACO[®]ped diabetic foot system, featuring a lightweight

but stable lattice frame structure together with the before mentioned vacuum cushion, offers the stability, of a cast. As a result, shear forces underneath the foot, as would normally occur when walking without cast-like protection, can be diminished. Unlike a total contact cast, the VACO[®]ped diabetic foot can be removed and reapplied instantly for wound control purposes and in order to accommodate possible structural changes to the bone system of the foot.



VACO[®]ped diabetic foot

A modular treatment for diabetic feet




Everything you need to ensure optimum treatment

- Vacuum cushion and rigid lattice frame structure for the fit and stabilization of a cast
- Dampening, dual-density insole
- Rocker bottom for pressure distribution and physiological walking gait
- Removable and washable inner liner for greater hygiene
- Optional: Security seals for patient control



Pressure peaks are effectively reduced with vacuum cushion

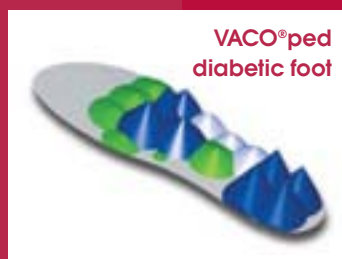

Rocker bottom for a physiological walking gait

VACO[®]ped
diabetic foot

The benefits of VACO[®]ped diabetic foot at a glance

- Promotes wound healing with even pressure distribution and reduction of shear forces by stabilizing the ankle joint
- Early mobilisation and physiological walking gait
- Rapid application and adjustment
- Easy access for wound control
- Improved quality of life – comfort, hygiene and mobility
- Economically and environmentally friendly – VACO[®]ped diabetic foot substitutes for a series of casts and saves time



Indication

Diabetic foot syndrome

Ordering Information

Two sizes, unilateral.

Sizes:

EU	S	35-39	UK	S	2 1/2 - 6	US	S	Men 5 - 7 / Women 7 - 9
	L/M	39-46		L/M	6 1/2 - 11		L/M	Men 7 - 13 / Women 9 - 11+

1. VACO[®]ped diabetic foot
2. VACO[®]pedes diabetic foot Replacement-Set

Standard for wound treatment
 Low-rise version, for ulceration at toes
 Contains new vacuum cushion, velour liner, insole and rocker bottom (continuous exchange of replacement kit is necessary after each 3-month period)

3. VACO[®]splint

For pre-/post-operative splinting

For more information, please visit www.vacodiaped.com

For prices and order information, please contact your local representative or OPED directly.



OPED GmbH
 Mühlbachweg 2
 D-83626 Valley
 Germany

OPED AG
 Sumpfstraße 5
 CH-6312 Steinhausen
 Switzerland

OPED Inc.
 5031 West W.I. Harris Blvd., Suite B
 Charlotte, NC 28269
 USA

Fon +49/8020/9079-0
 Fax +49/8020/9079-99
 Email mail@oped.de
www.oped.de

Fon +41/41/748 53 35
 Fax +41/41/748 53 40
 E-mail contact@oped.ch
www.oped.ch

Fon +1/704/596 2838
 Fax +1/704/596 2800
 E-Mail: contact@opedinc.com
www.opedinc.com